# TempoCemNE

## Handmix

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## Instructions for use

## English

TempoCemNE Handmix is a eugenol-free cement, based on zinc oxide, that is indicated for temporary cementation of crowns and bridges, inlays and onlays, as well as all types of temporary restorations. The 2 component material is mixed manually.

## Indication

- Temporary cementation of provisional and permanent crowns and bridges, inlays and onlays.

## Contra-indications

- Do not use the material in cases of allergies to any of the ingredients or, as can occur in rare cases, in the event of contact allergies.

## Basic safety instructions

- For dental use only!
- Keep out of reach of children!
- Avoid contact with skin and eyes! In the event of accidental contact, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Contains rosin

## Side effects

Isolated cases of hypersensitivity or contact allergy to components of the material cannot be excluded.

## Timing

0:00–0:30 min	Mixing
0:00–1:00 min	Mixing and handling
4:00–7:00 min	Curing completed

**Please note:** The times given apply for a room temperature of 23 °C and a normal relative air humidity of 50 %. Higher temperatures shorten, lower temperatures increase these times.

## Dosage and mixing

1. Measure out equal lengths of TempoCemNE catalyst and base paste (1:1).
2. Mix TempoCemNE catalyst and base paste for 30 seconds on a suitable mixing surface (e.g. a sheet of glass) using a plastic or metal spatula until the material develops an even consistency.

**Please note:** Make sure that you do not unintentionally mix up the lids of the catalyst and base pastes and that you do not mix up the two pastes, in order to avoid blocking of the tube openings caused by cross contamination.

## Recommended use

### Temporary cementation

1. Apply TempoCemNE to the dry temporary restoration.
2. Insert the temporary restoration within 1:00 min of start of mixing and attach to the dry, prepared tooth using slight pressure.
3. After curing, remove excess cement with a probe. It is easy to remove large chunks of excess cement.
4. Remove interdental excess by flossing carefully.  
The light color of the material makes it easy to distinguish from teeth.

## Storage and shelf life

- ▶ Store at room temperature (15 to 25 °C/59 to 77 °F) in a dry place!
- ▶ Protect against exposure to direct sunlight!
- ▶ Close the tube again after use!
- ▶ Do not use after the expiration date!

## Composition

---

- Catalyst paste: Natural resins, fatty acids, additives
- Base paste: Zinc oxide, paraffin, additives

## Classification

---

Complies with ISO 3107:1988

## Available product sizes

---

## Packaging

1 tube base paste @ 85g 1 tube catalyst paste @ 25g	REF 110137
--	------------

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