Luxatemp-Glaze & Bond

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Luxatemp-Glaze & Bond is a light-cured, single-component material based on multifunctional methacrylates for sealing the surface of and repairing temporary crowns and bridges. The varnished surface is high-gloss, protects against discolouration and produces a highly natural appearance.

Indication

- Surface sealing and repair of temporary crowns and bridges

Contraindication

- Do not use in the event of known allergies to one of the components (especially methyl methacrylates).

Please note

- If stored in a refrigerator (2 to 8 °C/ 36 to 46 °F), bring the material back to room temperature (15 to 25 °C/59 to 77 °F) before using.
- Light curing units should have an output of 450 nm and should be inspected regularly. Place the light as close as possible to the material.
- The light intensity should total at least 400 mW/cm².
- Luxatemp-Glaze & Bond bonds to teeth and sheets made of polycarbonate or polyacetate.
- Luxatemp-Glaze & Bond cannot be used as a dentine or enamel adhesive.
- Avoid contact with materials containing eugenol.
- Close the bottle immediately after every use.

Recommended use

Surface varnishing

1. Apply a thin layer of the material to the corresponding surfaces without air bubbles.
2. Cure each section for 20 seconds with a halogen lamp or > 10 seconds with an LED lamp (>1000 mW/cm²), or cure the entire restoration for at least 90 seconds in the light oven.

To ensure that no oxygen-inhibited layer forms, the material should be light cured within 2:00 minutes after use.

Priming for repair

1. Roughen the surface to be repaired with a bur.
2. Apply a thin layer of the material to the corresponding surfaces without air bubbles.
3. Expose to a halogen lamp for 20 seconds or to an LED lamp (>1000 mW/cm²) for ≥ 10 seconds.
4. Apply freshly mixed temporary material, e.g. Luxatemp Star, or a light-cure composite, e.g. LuxaFlow Star, to the corresponding surfaces. Follow the instructions regarding use of the material employed.

Storage

- ▶ Store in a dry place at 2 to 25° C (36 to 77° F).
- ▶ If stored in a refrigerator (2 to 8 °C/ 36 to 46 °F), bring the material back to room temperature (15 to 25 °C/59 to 77 °F) before using.

Composition

Multifunctional acrylates, methyl methacrylate, catalysts, stabilizers, additives

Presentation

Pack

1 bottle with 5 ml liquid, 25 brushes	REF 211927
---------------------------------------	------------

Accessories

50 brushes	REF 110041
------------	------------

Keep away from children. For use by dental professionals only.

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