LuxaForm

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Polycaprolacton, Additive, Pigmente

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Instructions for use

English

LuxaForm is an easy-to-use thermoplastic impression polymer. LuxaForm is the ideal solution for patients with a small oral aperture, strong gag reflex, or when the back molars are being prepared.

Indication

- To take impressions for temporary restorations

Recommended use

1. Use one LuxaForm disc for each tooth and place in hot water at about 70 °C until the material has become soft and transparent (after about 1 minute). Use

several discs when taking impressions of more than one unit. The discs can be kneaded together easily when in a transparent state.

2. Preform transparent material into a cube, if necessary, and model around the teeth.

The material rapidly returns to a rigid and opaque blue state after cooling. Cooling can be further hastened by spraying briefly with water.

3. Remove hardened LuxaForm from the tooth and insulate lightly with vaseline or another separating agent. Apply temporary crown and bridge material (e.g. Luxatemp Star) as usual.

Please note

- Soft LuxaForm adheres to gloves, so it should always be kneaded without gloves. When placing in the mouth, insulate the glove fingertips lightly with vaseline.
- In order to ensure a safe removal, do not mould LuxaForm too tightly against the teeth when taking an impression.
- Block any undercuts with a suitable material, e.g. wax.
- Do not use with MMA-containing crown and bridge materials.
- Do not use LuxaForm for impressions of more than 5 teeth.
- Any further modelling of the impression should be done with a scalpel.

Composition

Polycaprolactone, additives, pigments

Storage

- ▶ Store in a dry place at room temperature (15 to 25 °C/59 to 77 °F).
- ▶ Do not use after the expiry date.

LuxaForm impressions are stable for months and can also be reused later, e.g. if a temporary restoration breaks.

Presentation

Pack of 72 tablets	REF 110415
--------------------	------------

Keep away from children. Only for use by dental professionals!

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Composition

Polycaprolactone, additifs, pigments

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