Silagum-Mono

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Klassifikation

ISO 4823:2000, Type 2: Medium-bodied consistency

Handelsformen

Nachfüllpackung	
2 Kartuschen à 50 ml Paste, 4 Automix-Tips, 4 Intraoral-Tips	REF 909718

Vorteilspackung	
8 Kartuschen à 50 ml Paste, 16 Automix-Tips, 16 Intraoral-Tips	REF 909719
8 Tuben à 135 ml (170 g) Paste, 2 Mischblöcke, 2 Spatel	REF 909427

Instructions for use

English

Product description

Monophase impression material based on A-silicones. The material is available in the Automix safety cartridge and as a hand-mixed variant.

Indications

- Crown and bridge impressions, inlay and onlay impressions
- Functional impressions

Basic safety instructions

- ▶ For dental use only!
- ▶ Keep out of reach of children!

Interactions

- ▶ Never allow the impression to come into contact with organic solvents or liquids containing solvents as this may result in a swelling of the material and may therefore lead to inaccuracies in the cast.
- ▶ Avoid contact with retraction solutions, because in some instances residues of retraction solutions can increase the setting time.
- ▶ Do not mix Silagum-Mono with condensation curing silicone or polyether impression materials.
- ▶ Avoid contact between the impression material and latex gloves because these prevent the setting process. In the event of prior contact between the hands and latex gloves, wash hands thoroughly. It is recommended that PVC or PE based gloves be used.

Using the Automix cartridge

Insert cartridge

1. Push up the lever [Fig. 1, Pos. A] at the back of the Automix dispenser and pull back the slide [Fig. 1, Pos. B] completely.
2. Lift up the plastic lever [Fig. 1, Pos. C], insert the cartridge and lock it using the plastic lever [Fig. 1, Pos. C].

Please note: Make sure that the notches on the cartridge line up with those on the Automix dispenser.

Attach mixing tip

1. Remove the cartridge cap [Fig. 1, Pos. D] or used mixing tip by turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [Fig. 1, Pos. E].

Please note: In order to attain an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are also available as refill packs.

Please note: Make sure that the notches on the mixing tip line up with those on the cartridge.

3. Turn the mixing tip clockwise 90° to lock it.

Applying the material

- ▶ When using a cartridge for the first time, squeeze out a pea-sized amount of material and discard it.
- ▶ The material is mixed in the mixing tip by pulling the trigger [Fig. 1, Pos. F] of the Automix dispenser and it is then applied directly.

Please note: After the application, leave the used mixing tip on the cartridge to cap unused material!

Remove cartridge

1. Push up the lever [Fig. 1, Pos. A] at the back of the Automix dispenser and pull back the slide [Fig. 1, Pos. B] completely.
2. Lift up the plastic lever [Fig. 1, Pos. C] and remove the cartridge.

Handling the hand-mixing system

1. Measure out equal lengths of catalyst and base paste (1:1) at room temperature.
2. Mix the catalyst and base paste for 30 seconds on a suitable mixing surface (e.g. a sheet of glass) using a plastic or metal spatula until the color is homogenous.
3. Fill the mixed material into an elastomer syringe for application.

Please note: Close the tube again immediately after use! Make sure that you do not unintentionally mix up the lids of the catalyst and base pastes and that you do not mix up the two pastes, in order to avoid blocking of the tube openings caused by cross contamination.

Instructions for use

- ▶ Silagum-Mono can be used in all common non-perforated impression trays. In order to improve the adhesion to metal or plastic trays, a tray adhesive can be applied.
- ▶ For the individual times refer to the »Technical data« section.

Recommended use

1. Fill the tray with Silagum-Mono. Whilst the assistant fills the tray, the dentist can syringe the preparation with Silagum-Mono. In order to do so, add some of the material to an elastomer syringe.

Please note: The filling of the impression tray and syringing should be completed at the same time, because the higher temperature in the mouth leads to a quicker setting of the syringed material compared to the tray material.

Please note: To avoid air bubbles, the end of the mixing tip should stay immersed in the impression material throughout the application process.

2. Position the impression tray in the mouth within the working time and allow the impression to form.
3. Remove the impression from the mouth once the setting time has ended.

Removal of the impression

- ▶ Remove the impression parallel to the longitudinal axis of the prepared teeth.
- ▶ After removal, proceed as described in the section »Cleaning and disinfection of the final impression«.

Cleaning and disinfection of the final impression

- ▶ Only disinfect the final impression using dipping solutions. DMG recommends the use of a 2% glutaraldehyde solution.
- ▶ Please ensure that the disinfectant used is indicated for use with addition curing silicone impression material.
- ▶ Follow the instructions of the disinfectant solution's manufacturer precisely.
- ▶ Leave the final impression to dry and store until casting, see »Storage of the impression«.
- ▶ For more information, refer to the comments of the DGZMK. Comments of the DGZMK 3/93 V 1.0; www.dgzmk.de.
- ▶ ADA recommends following Council on Dental Materials, Instruments and Equipment. Disinfection of impressions. JADA 1991;122(3):110; www.ada.org

Casting and galvanization

- ▶ The impression should be cast 30 minutes at the earliest after removal from the mouth.

DMG recommends using special purpose hard plaster (ISO 6873; Type 3, e.g. Moldano, Bayer) for the cast and an ultra hard dental plaster for the forms (ISO 6873, Type 4; e.g. Fujirock II, GC). The impression can be copper or silver-plated.

Storage of the impression

- ▶ Store at room temperature (15 – 25 °C/59 – 77 °F) in a dry place!
- ▶ Protect against exposure to direct sunlight!

The impression is stable and can be cast even after months without loss of accuracy.

Technical data

Silagum-Automix Mono	
Working time*	≤ 1:30 min
Recommended time in mouth	≥ 2:30 min
Dimensional change after 24 h	≤ 0.5%
Compression set	≤ 0.5%
Strain in compression	≤ 3.5%

Silagum-Handmix Mono	
Mixing time	0:30 min
Working time*	≤ 2:30 min
Recommended time in mouth	≥ 3:30 min
Dimensional change after 24 h	≤ 0.35%
Compression set	≤ 0.4%
Strain in compression	≤ 4.0%

* The times apply at a room temperature of 23 °C and a normal relative air humidity of 50%. Higher temperatures shorten, lower temperatures increase these times.

Disinfection of the Automix dispenser

The Automix dispenser is reusable after disinfection, although it should be replaced if visible signs of wear appear.

Disinfection can be carried out using a commercially available dipping disinfectant. DMG recommends using only disinfecting solutions listed by the RKI (Robert Koch Institute). The Automix dispenser is also autoclavable.

Storage and shelf life

- ▶ Store at room temperature (15 – 25 °C/59 – 77 °F) in a dry place!
- ▶ Do not store in the fridge or freezer.
- ▶ Protect against exposure to direct sunlight.
- ▶ After the application, leave the used tip on the cartridge to cap unused material.
- ▶ Do not use after the expiration date.

Composition

Addition curing vinyl polysiloxanes, hydrogen polysiloxanes, fillers, pigments, additives, platinum catalyst

Classification

ISO 4823:2000, type 2: medium-bodied consistency

Available product sizes

Refill packs	
2 cartridges @ 50 ml paste, 4 Automix tips, 4 Intraoral tips	REF 909718

Economy pack	
8 cartridges @ 50 ml paste, 16 Automix tips, 16 Intraoral tips	REF 909719
8 tubes @ 135 ml (170 g) paste, 2 mixing blocks, 2 spatulas	REF 909427

Federal law (USA) restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

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