Silagum®-Putty

Silagum®-Light

904439/04.04

Gebrauchsinformation

Deutsch

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Instructions for use

English

Pre-impression material (Silagum-Putty) and correction impression material (Silagum-Light) based on A-silicones for all two-phase impression techniques. Silagum-Light is available in two different application forms: in the Automix cartridge and in a handmix version.

Indication

Crown and bridge impressions, inlay and onlay impressions

Mixing of the materials

Silagum-Putty

Mix equal parts in volume or weight of base paste and catalyst paste at room temperature. Take the base paste with the coloured spoon and the catalyst paste with the white spoon. Knead the components with the fingertips for at least 30 seconds until the colour is homogeneous. Close jars immediately after use. Do not interchange caps of jars.

Do not wear latex gloves when mixing and kneading the material. If you had contact with latex gloves before mixing, wash your hands thoroughly. The use of PVC or PE gloves is recommended.

Silagum-Light

Handling

Automix: Silagum-Automix Light is automatically dispensed and mixed when extruding the two components out of the mixing tip. When using a new cartridge, remove and discard the cap. Attach the mixing tip and, if need be, an Intraoral-Tip to the cartridge. Apply the material. Leave the used mixing tip on the cartridge after use. It serves as a cap until next use.

Handmix

At room temperature, using a plastic spatula, mix equal parts in volume or weight of base paste and catalyst paste for at least 30 seconds to a homogenous colour. Mixing the material to a homogenous colour will ensure that the material is completely mixed. For application, load the mixed material into an elastomer syringe and apply. Close tubes immediately after use. Do not interchange caps of tubes.

Recommended Use

Impression Tray

Silagum-Putty may be used in all common, non-perforated impression trays. To get better adhesion to metal or plastic trays, a tray adhesive may be applied.

Impression Taking

The tray is loaded with Silagum-Putty and introduced into the mouth within working time. The position must be kept in the mouth for the recommended time.

Silagum-Light is applied directly onto the tray and/or the teeth. The tray is inserted in the mouth within working time and removed after the material has set. Intraoral-tips are included for direct, intrasulcular application around the preparation.

For individual times please consult the table.

Do not use Silagum together with condensation curing or polyether type impression materials.

Special Techniques

Double Mixing Technique

The tray is loaded with Silagum-Putty. While the tray is being loaded by the assistant, the dentist can begin to syringe the preparations with Silagum-Light. Loading of the tray and syringing should be completed at the same time as the higher temperature in the mouth leads to a quicker set of the wash impression material as compared to the tray material. Position the loaded tray in the mouth and take the impression. After setting, the tray can be removed.

Correction Impression

Generous pressure relief channels are cut in the pre-impression and imperfections are removed. After repositioning, the impression is carefully cleaned and dried. Apply Silagum-Light around the preparation and onto the pre-impression. Insert the tray into the mouth and exert a short pressure. After setting, the tray can be removed.

To ensure that both materials bond properly, no more than 30 minutes should pass between the impression and the preliminary impression. The double mixing technique may be used alternatively.

Removal

Remove the impression parallel to the longitudinal axis of the prepared teeth. After removal, thoroughly clean with water and disinfect the impression.

Disinfection

Use only dipping solutions for the disinfection of the impression. Recommended is the use of a 2% glutaraldehyde solution. Please ensure that the used disinfectant is indicated for the use with addition curing silicone impression material.

After removal from the mouth, clean impression under running water for 15 seconds. Leave the impression in the disinfecting solution for about 10 to 15 minutes. Then rinse off the disinfection solution for 15 seconds under running water. For more information, refer to the comments of the DGZMK. Comments of the DGZMK 3/93 V 1.0; www.dgzmk.de

ADA recommends following Council on Dental Materials, Instruments and Equipment. Disinfection of impressions. JADA 1991; 122(3): 110; www.ada.org

Casting and Galvanization

After disinfection, dry impression and store at normal room temperature (max. 78°F / 25°C). Impression should be poured 30 minutes after removal from the mouth at the earliest. We recommend using special purpose hard plaster (ISO 6873, Type 3; e.g. Moldano, Bayer) for the cast and an ultra hard dental plaster for forms (ISO 6873, Type 4; e.g. Fujirock II, GC).

The impressions can be copper- or silverplated.

Storage of the Impression

Store impression at normal room temperature (max. 78°F / 25°C). The impressions are stable and can be poured even after months without loss of accuracy.

Please Note

- Never allow the impression to come into contact with organic solvents or liquids containing same as this may result in swelling of the material and therefore may lead to inaccuracies.
- Possible residues of retraction aids can increase the setting time.
- Never allow the impression material to come into contact with latex gloves. Use of PVC or PE gloves is recommended.

Composition

Addition curing vinylpolysiloxanes, hydrogen polysiloxanes, filler, pigments, additives, platinum catalyst

Storage

Store and use in dry location at normal room temperature not exceeding 78°F / 25°C. We recommend not storing material in the refrigerator or in the freezer. Leave used mixing tip on the cartridge after use. It serves as a cap. Close tubes and jars immediately after use. Do not use after expiration date.

Technical Data

Silagum-Putty	Fast	Standard	Soft
Mixing time	0:30 Min	0:30 Min	0:30 Min
Total working time* (incl. mixing)	≤ 1:15 Min	≤ 1:45 Min	≤ 1:45 Min

Recommended time in mouth	≥ 2:30 Min	≥ 3:30 Min	≥ 3:30 Min
Dimensional change (24h)	≤ 0.5%	≤ 0.5%	≤ 0.5%
Compression set	≤ 0.7%	≤ 0.7%	≤ 0.7%
Strain in compression	= 1.7%	= 1.7%	= 2.5%

Silagum-Light	Fast	Standard
Mixing time	-	0:30 Min
Total working time* (incl. mixing)	≤ 1:30 Min	≤ 2:15 Min
Recommended time in mouth	≥ 2:15 Min	≥ 3:30 Min
Dimensional change (24h)	≤ 0.4%	≤ 0.4%
Compression set	≤ 0.4%	≤ 0.4%
Strain in compression	= 3.5%	= 3.5%

* Times refer to normal room temperature (72 °F / 23°C and a normal relative humidity of air (50%). Higher temperatures shorten, lower temperatures increase those times.

Classification

Silagum-Putty: ISO 4823:2000, Type 0: Putty consistency

Silagum-Light: ISO 4823:2000, Type 3: Light-bodied consistency

Packaging

Silagum-Putty

2 jars @ 262 ml paste	
Silagum-Putty Standard	REF 909018
Silagum-Putty Soft	REF 909019
Silagum-Putty Fast	REF 909037

Economy Pack

8 jars @ 262 ml paste	
Silagum-Putty Standard	REF 909405
Silagum-Putty Soft	REF 909036
Silagum-Putty Fast	REF 909038

Super Saver Pack

16 jars @ 262 ml paste	
Silagum-Putty Standard	REF 909420
Silagum-Light	

Refill

2 cartridges @ 50 ml g paste, 12 Automix-Tips, 12 Intraoral-Tips	
Silagum-Light	REF 909713
Silagum-Light Fast	REF 909714
2 tubes @ 90 ml (110 g) paste	
1 mixing pad, 1 spatula	REF 909373

Economy Pack

8 cartridges @ 50 ml paste, 50 Automix-Tips, 50 Intraoral-Tips	
Silagum-Light	REF 909715
Silagum-Light Fast	REF 909720
8 tubes @ 90 ml (110 g) paste, 2 mixing pads, 1 spatula	
Silagum-Handmix	REF 909422

Federal Law (USA) restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

Keep away from children! For dental use only!

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